

Dentemp® Recap-it™

Cap & Crown Repair

Dentemp® Recap-It™ Cap & Crown Repair is a unique, no mix cement made especially for replacing loose caps and crowns. Tested, safe and effective.

⚠️ READ DIRECTIONS CAREFULLY BEFORE USING. KEEP OUT OF REACH OF CHILDREN.

CAUTION: This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a temporary remedy. See your dentist as soon as possible for permanent work to be performed. Failure to do so may cause infection and/or serious injury. Store at room temperature with tube closed to keep material soft. Lot code on container.

Directions: Wash hands. Prepare crown by carefully removing as much old cement as possible from inside the crown. Rinse well. Clean and rinse tooth area with water and leave moist. Fit the crown back on the tooth before applying new material. Make sure it fits properly. If you cannot do this, **DO NOT PROCEED.** See your dentist. Make sure inside of crown is dry. Open tube by reversing the cap and punching a hole in the tip of the tube. Apply a small amount of material onto the upper inside edge of the crown. Remoisten tooth area, place crown on wet tooth and press firmly. Bite down several times to ensure a proper fit. If the bite is not completely comfortable, remove and repeat all steps until the crown feels normal. **WAIT 5 MINUTES AND THEN GENTLY REMOVE ANY EXCESS MATERIAL. RINSE OUT MOUTH WELL WITH WATER.** Allow at least 1 hour to set before eating. Cement will fully set in 1 to 3 hours. **NOTE:** When replacing a bridge, apply material sparingly to only 1 hollow tooth at each end of the bridge. Dentemp Recap-It Cap & Crown Repair should not be used on implants or posts without dentist's advice.

Ingredients: Calcium Sulfate, Dimethoxytetraethylene Glycol, Zinc Oxide, Potassium Sulfate, Ethylmethacrylate Polymer, Triacetin.

833-362-2763 | consumeraffairs@doc-brands.com
www.doc-brands.com

©2018 DOC Brands, Inc. Wayne, PA 19087 USA